Rollator Series H1010

applies to H1010 - H1019

User instructions

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Mode d'emploi

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Bruksanvisningar

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 helavo

User Instructions

4		USER INSTRUCTIONS
22		BEDIENUNGSANLEITUNG
40		MODE D'EMPLOI
58		INSTRUCCIONES DE USO
76		ISTRUZIONI PER L'USO
94		BRUKSANVISNINGAR
112		GEBRUIKSAANWIJZING
130		INSTRUKCJA OBSŁUGI

Rollator Series H1010

English user instructions

Intended use

This rollator is intended to be used by persons with reduced mobility as a walking aid and for transporting smaller objects. This rollator model is suitable for both indoor and outdoor environments. It is intended for use on solid ground by persons with a maximum weight of 300 lbs. Do not use the rollator in combination with other mobility aids.

The user should have the necessary physical, visual and cognitive abilities to operate the rollator safely.

Before using your rollator

Read the user instructions carefully **before** assembling and using the rollator and keep it in a safe place. Consult your medical professional to determine if the rollator is suitable for your needs.

Our rollator is delivered pre-assembled. You can assemble it on your own in a few steps. Assembly takes about 10 minutes. No tools are needed for assembly. Put a blanket underneath to protect your floor.

The rollator can be used by different people. Before use, check that the height is set correctly.

Important: Please check the rollator for transportation damage. Do not use the rollator if there is obvious damage. Contact us regarding repairs or return.

If you have any questions regarding the assembly, the rollator or if you are unsure about anything, please contact our customer service at the e-mail address: support@helavo-rollators.com

Scope of delivery

The Helavo Series H1010 comes with a frame and the following parts:

- Two cane holders
- Multifunctional bag
- Wing screws
- Bolt
- Attachment Strap
- Step aid
- User instructions
- Product inserts

Description of the rollator

H1010-Series rollators

Helavo model series H1010 consists of the following rollators:

- H1010: Allround-Rollator, silver
- H1011: Allround-Rollator, black
- H1012: Allround-Rollator, red
- H1013: Allround-Rollator, blue

Symbols

Manufacturer

Date of manufacture

Catalog No.

Batch code

Medical Device


Unique device identifier

Instruction for use

CE-Marking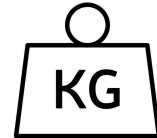

Maximum load capacity

Safety

Warning: The rollator can provide assistance to a person with a maximum weight of 300 lbs.

Follow the safety instructions

1. Make sure the parking brake ("G") is active when the rollator is parked.
2. Check the handle height and make sure the spring buttons ("H") are locked before use.
3. Make sure the rollator is completely unfolded and locked into position with a "click" sound.
4. Keep your hands away from the folding mechanism to avoid entrapment.

What not to do

1. Do not use the rollator as a wheelchair.
2. Do not sit on the rollator if the parking brake ("G") is not active or if it is at an angle.
3. Do not "drop" onto the seat ("B").
4. Do not hang any objects on the brake handles ("G") or hand grips ("F").
5. Do not stand on the rollator or use it as a stepladder.

If you have any questions regarding the assembly, the rollator or if you are unsure about anything, please contact our customer service at the e-mail address: support@helavo-rollators.com

Assembly instructions


1. Remove the rollator from the packaging and remove the packaging material.

2. Remove the orange attachment strap. The strap can be reused after final assembly to secure the rollator in its folded position.

3. Lay the rollator down so that the backrest ("A") is facing up. Loosen the wing screws ("D") on the front wheels ("E").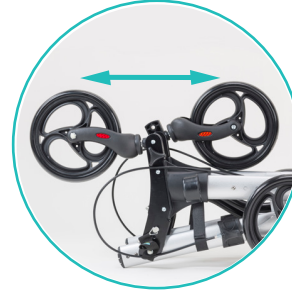


4. Unfold the front wheels ("E"). Secure the wheels with the previously loosened wing screws ("D").

Assembly instructions

5. Turn the rollator so that the handles ("F") are facing up. Pull the bracket off the bolt ("I") from the inside of the rear wheel and pull the bolt ("I") out.

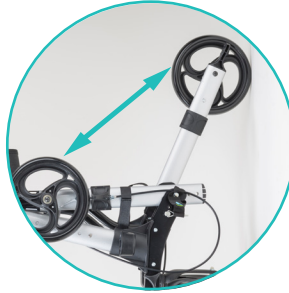6. Unfold the rear wheel axle of the rear wheel ("K").

7. Fasten the rear wheel axle ("K") with the previously loosened bolt ("I"). To do this, insert the bolt into the provided hole.

8. Fasten the bracket over the unfolded rear wheel axle ("K"). Then pull the bracket over the end of the bolt ("I"). Repeat steps 5 - 8 for the second rear wheel.


Assembly instructions

9. Check that the two bolts ("I") are properly secured and that the wing screws ("D") are securely tightened.

10. Put the rollator in an upright position and pull it apart by both handles ("F"). Push it down on both sides of the seat ("B") as shown.
Important: For the rollator to be fully unfolded, it must lock into place with a "click" sound.

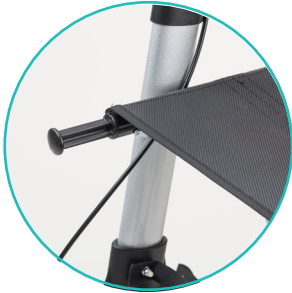11. Attach the multifunctional bag ("C").

12. Hook the tabs of the multifunctional bag ("C") to the brackets provided.

Adjusting the height

Press the spring button ("H") on the frame of the rollator and adjust the handle to the appropriate height. Repeat for the other side. Make sure the spring buttons on both handles are locked. Make sure that both handles are set to the same height.

Handle height: 31.1 - 37.4 inch

Your ideal handle height

Adjust the handles ("F") of the rollator so that they are at the height of your wrist when your arms are hanging. When you place your hand on the handle, your elbow should be slightly bent. If you find that you need to bend over or keep your arms bent at a right angle, adjust the height of the handles.

Brakes


1. Pull the brake handles ("G") up to brake the rollator.

2. Push the brake handles ("G") down to lock the rear wheels. In this position, the parking brake of the rollator is active.
To release the parking brake again, pull the brake handles ("G") upward.

Adjusting the brakes

To adjust the rear brake ("L"), adjust the adjustment screw and locking nut. These are located on the rear wheels ("K") of the rollator. Turn the adjustment screw counterclockwise to increase the tension. Then secure the adjustment by turning the locking nut clockwise. Hold the adjustment screw in place while doing so.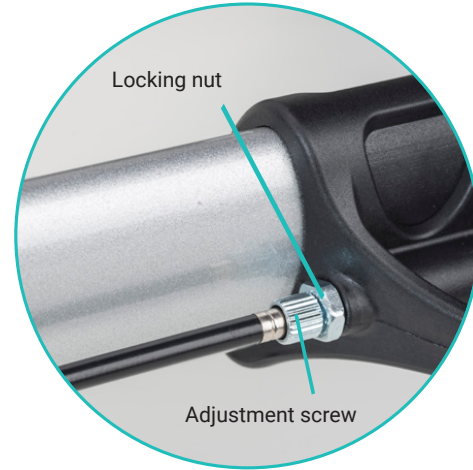

! Important

If the brake adjustment has been made so that the rear brake ("L") is set too loose or too tight, this will affect the function of the brake. If the brake is set too tight, it could drag on the wheel. If the brake is set too loose, it could affect the operation of the brake.

Removable cane holders

1. Attach the cane holders ("J") to both sides of the frame in the provided positions.

2. To do this, slide the cane holder ("J") onto the holder from above.

Folding and storing the rollator

1. Pull the handle in the middle of the seat ("B") firmly upwards to fold the rollator.

2. Continue to push the rollator on both sides of the frame.

3. Secure the rollator with the orange attachment strap at the brackets where the multifunctional bag ("C") hangs.

Transportation of the rollator

The bag included in the delivery is a multifunctional bag ("C").

Open the zipper on the lower part of the multifunctional bag so that you can open it completely. Then pull the multifunctional bag apart. Follow these steps in reverse order to shrink the multifunctional bag back down and attach it to your rollator.

Transportation of the rollator

Set the height of the handles ("F") to the lowest possible level to prepare the rollator for transportation. (Also see chapter "Adjusting the height"). Then follow steps 1 - 3 in chapter "Folding and storing the rollator".

1. Remove the bolt ("I") and fold in the rear wheels ("K"). Then push the bolts back through the holes provided and fasten the brackets (Also see steps 5 - 8 in the chapter "Assembly instructions" in reverse order).

2. To store the rollator in the multifunctional bag ("C"), loosen the wing screws ("D") of the front wheels ("E"). Fold in the front wheels. Fasten the front wheels with the previously loosened screws (Also see steps 3 and 4 in the chapter "Assembly instructions" in reverse order).

Maintenance

1. Check the rollator at regular intervals for loose wing screws ("D") and defective spring buttons ("H").
2. Periodically check for proper operation of the rear wheel brakes ("L").
3. Replace damaged or worn parts.

Cleaning

The rollator can be cleaned with a damp cloth / sponge and warm soapy water. If necessary, you can use an antibacterial surface cleaner. Dry the areas thoroughly afterwards. When cleaning, take care not to leave any residue on the seat and handles so that the surface does not become slippery in these areas. Do not use scouring pads or scrapers to avoid damaging the surface coating.

Use and disposal

It is important that the rollator is inspected and serviced regularly to ensure that it is in good condition and safe to use.

Be aware of the following points:

1. Do not use the rollator if any parts are defective or missing. Especially screws, wheels or spring buttons ("D"/"E"/"H"/"K").
2. Do not use the rollator if the wheels ("E"/"K") are defective.
3. Do not use the rollator if the height adjustment ("H") of the handles does not engage properly or is defective.
4. Do not use the rollator if the brake ("G"/"L") does not work properly.
5. Do not use the rollator if the bolt ("I") is missing.
6. Do not use the rollator if it does not engage properly when folded up.

Disposal

The disposal of a rollator should be done carefully to ensure that no pollution occurs and that all materials are disposed of properly.

Check the local regulations in your city or town to find out how to properly dispose of the rollator.

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