# Rollator Walker H1020

applies to H1020 - H1029

---

User instructions

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Mode d'emploi

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Bruksan isningar

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 helavo

## User Instructions

4		USER INSTRUCTIONS
23		BEDIENUNGSANLEITUNG
42		MODE D'EMPLOI
61		INSTRUCCIONES DE USO
80		ISTRUZIONI PER L'USO
99		BRUKSANVISNINGAR
102		GEBRUIKSAANWIJZING
105		INSTRUKCJA OBSŁUGI

# Rollator Series H1020

## English User instructions

### Intended use

This rollator is intended to be used by persons with reduced mobility as a walking aid and for transporting smaller objects. This rollator model is designed for indoor and outdoor use. It is intended for use on solid ground by persons with a maximum weight of 300 lbs. Do not use the rollator in combination with other mobility aids.

The user should have the necessary physical, visual and cognitive abilities to operate the rollator safely.

### Before using your rollator

Read the user instructions carefully **before** assembling and using the rollator and keep it in a safe place. Consult your medical professional to determine if the rollator is suitable for your needs.

You can assemble this rollator on your own in a few steps. Assembly takes about 10 minutes. No tools are needed for assembly. Put a blanket underneath to protect your floor.

The rollator can be used by different people. Before use, check that the height is set correctly.

**Important:** Please check the rollator for transportation damage. Do not use the rollator if there is obvious damage. Contact us regarding repairs or return.

If you have any questions regarding the assembly, the rollator or if you are unsure about anything, please contact our customer service at the e-mail address: [support@helavo-rollators.com](mailto:support@helavo-rollators.com)

## Scope of delivery

The Helavo Series H1020 comes with a frame and the following parts:

- Removable cup holder
- Removable cane holder
- Adjustable Backrest
- Bag
- Wing screws
- Front wheels
- Rear axle with rear wheels
- Velcro strap
- User instructions
- Product insert
- Step aid

## Description of the rollator

## H1020-Series rollators

Helavo model series H1020 consists of the following rollators:

- H1020: AllroundRollator, black

## Symbols

Manufacturer

Date of manufacture

Catalog No.

Batch code

Medical Device


Unique device identifier

Instruction for use

CE-Marking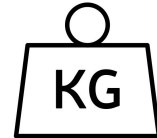

Maximum load capacity

## Safety

**Warning:** The rollator can provide assistance to a person with a maximum weight of 300 lbs. The maximum load of the bag of 11 lbs should not exceeded.

### Follow the safety instructions

1. Make sure the parking brake ("H") is active when the rollator is parked.
2. Check the handle height and make sure the height adjustment ("K") is locked before use.
3. Make sure the rollator is completely unfolded and locked into position with a "click" sound.
4. Keep your hands away from the folding mechanism to avoid entrapment.

### What not to do

1. Do not use the rollator as a wheelchair.
2. Do not sit on the rollator if the parking brake ("H") is not active or if it is at an angle.
3. Do not "drop" onto the seat ("D").
4. Do not hang any objects on the brake handles ("H") or handgrips ("A").
5. Do not stand on the rollator or use it as a stepladder.

If you have any questions regarding the assembly, the rollator or if you are unsure about anything, please contact our customer service at the e-mail address: [support@helavo-rollators.com](mailto:support@helavo-rollators.com)

## Assembly instructions

1. Remove the rollator from the packaging and remove the protective foils. Spread out the parts for assembly as shown. The seat ("D") of the rollator lies flat.

2. Remove the two wing screws ("N") on the frame of the rollator at the "RB" mark.

3. Now insert the rear wheel axle with rear wheel ("M") with the marking "RB" into the frame at the corresponding marking "RB". Both markings point to the outer side of the rollator.

4. **Important:** Make sure that the pictured marking on the rear wheel axle with rear wheel ("M") is flush with the frame. This way, the wing screw holes ("N") will be flush with those on the frame.

## Assembly instructions

5. Fasten the rear wheel axle with rear wheel ("M") with the wing screws ("N") loosened before.

6. Now repeat steps 2. - 5. for the second rear wheel axle with rear wheel ("M") with mark "LB" on the other side of the frame.

7. Remove the wing screw ("F") on the front wheel ("G") with the marking "RF".

8. Insert the front wheel ("G") into the frame at the corresponding "RF" mark.

## Assembly instructions

9. Fasten the front wheel ("G") with the previously loosened wing screw ("F"). The gray wing screw pointing inwards. When turning, the screw head will tighten in the frame.

10. Repeat steps 6. - 8. for the second front wheel ("G") with the mark "LF" on the other side of the frame.

11. Put the rollator in an upright position and pull it apart by both handles ("A"). Push it down on both sides of the seat ("D") as shown. Important: For the rollator to be fully unfolded, it must lock into place with a "click" sound.

12. Attach the backrest ("C") to both sides of the handles ("A") and fix it by tightening the grey wing screws ("J"). The Helavo logo should be facing the front of the rollator. Tip: After assembly, you can adjust the backrest to your desired height.

## Assembly instructions

- 13.** Press and hold the gray height adjustment button ("K") and insert the two handles ("A") into the frame one after the other. Adjust the handles so that both sides are the same height.

- 14.** Our recommendation: Fix the brake cables on both sides of the frame using the velcro tape as shown. Be careful not to cover any reflectors when doing this.

- 15.** Attach the bag ("E"). Hook the tabs of the bag into the hooks provided.

## Adjusting the height

Press the gray button on the height adjustment ("K") on the frame of the rollator and adjust the handle to the appropriate height. Repeat for the other side. Make sure the gray buttons on both handles are locked. Make sure that both handles are set to the same height.

**Handle height:** 30.9 - 35.8 inch

## Your ideal handle height


Adjust the handles ("A") of the rollator so that they are at the height of your wrist when your arms are hanging. When you place your hand on the handle, your elbow should be slightly bent. If you find that you need to bend over or keep your arms bent at a right angle, adjust the height of the handles.

## Brakes


1. Pull the brake handles ("H") up to brake the rollator.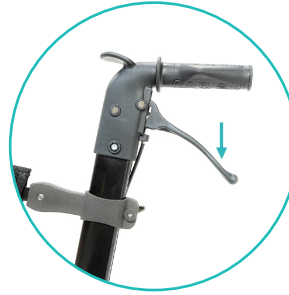

2. Push the brake handles ("H") down to lock the rear wheels. In this position, the parking brake of the rollator is active.  
To release the parking brake again, pull the brake handles ("H") upward.

## Adjusting the brakes

To adjust the rear brake ("O"), adjust the adjustment screw and locking nut. These are located on the rear wheels ("M") of the rollator. Turn the adjustment screw counterclockwise to increase the tension. Then secure the adjustment by turning the locking nut clockwise. Hold the adjustment screw in place while doing so.

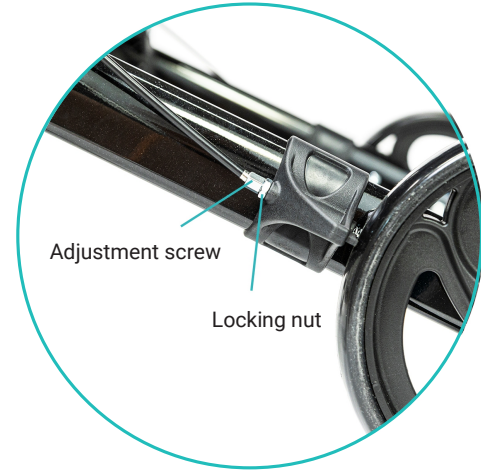### **!** Important

If the brake adjustment has been made so that the rear brake ("O") is set too loose or too tight, this will affect the function of the brake. If the brake is set too tight, it could drag on the wheel. If the brake is set too loose, it could affect the operation of the brake.

## Removable cane holder

1. Attach the cane holder ("L") to the side of the frame in the provided position.

2. To do this, slide the cane holder ("L") onto the holder from above.

## Removable cup holder

1. Attach the cup holder ("B") to the side of the handle in the provided position.


2. To do this, slide the cup holder ("B") onto the holder from above.

## Folding and storing the rollator


1. Pull the handle in the middle of the seat ("D") firmly upwards to fold the rollator.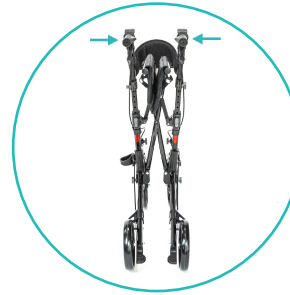

2. Continue to push the rollator on both sides of the frame.

## Adjusting the backrest

Press the gray width adjustment button ("I") on the backrest ("C") of the rollator and adjust the backrest to the appropriate width. Repeat for the other side. Make sure the width adjustment buttons on both sides are locked. Make sure that both sides are set to the same height.

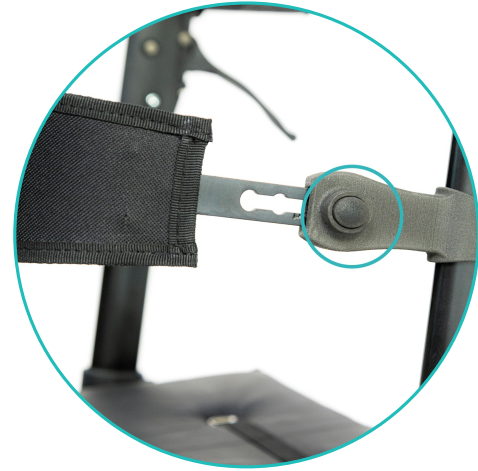## Maintenance

1. Check the rollator at regular intervals for loose wing screws ("F" / ""J" / "N") and defective height adjustments ("K").
2. Periodically check for proper operation of the rear wheel brakes ("O").
3. Replace damaged or worn parts.

## Cleaning

The rollator can be cleaned with a damp cloth / sponge and warm soapy water. If necessary, you can use an antibacterialsurface cleaner. Dry the areas thoroughly afterwards. When cleaning, take care not to leave any residue on the seat and handles so that the surface does not become slippery in these areas. Do not use scouring pads or scrapers to avoid damaging the surface coating.

## Use and disposal

It is important that the rollator is inspected and serviced regularly to ensure that it is in good condition and safe to use.

### Be aware of the following points:

1. Do not use the rollator if any parts are defective or missing. Especially screws or wheels ("F"/"G"/"M"/"N").
2. Do not use the rollator if the wheels ("G"/"M") are defective.
3. Do not use the rollator if the height adjustment ("K") of the handles does not engage properly or is defective.
4. Do not use the rollator if the brake ("H"/"O") does not work properly.
5. Do not use the rollator if it does not engage properly when folded up.

### Disposal

The disposal of a rollator should be done carefully to ensure that no pollution occurs and that all materials are disposed of properly.

Check the local regulations in your city or town to find out how to properly dispose of the rollator.

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