Rollator Series H1030

applies to H1030 - H1039

User instructions
Bedienungsanleitung
Mode d'emploi
Instrucciones de uso
Istruzioni per l'uso

 helavo

User Instructions

4		USER INSTRUCTIONS
22		BEDIENUNGSANLEITUNG
40		MODE D'EMPLOI
58		INSTRUCCIONES DE USO
76		ISTRUZIONI PER L'USO
94		BRUKSANVISNINGAR
112		GEbruIKSAANWIJZING
130		INSTRUKCJA OBSŁUGI

Rollator Series H1030

English user instructions

Intended use

This rollator is intended for use by persons with reduced mobility as a walking aid and for transporting smaller objects. This rollator model is specially designed for indoor use. It is intended for use on solid ground with a maximum load capacity of 240 lbs. Do not use the rollator in combination with other mobility aids.

The user should have the necessary physical, visual and cognitive abilities to operate the rollator safely.

Before using your rollator

Read the instruction manual carefully **before** assembling and using the rollator and keep it in a safe place. Consult your medical professional to determine if the rollator is suitable for your needs.

You can assemble this rollator on your own in a few steps. Assembly takes about 10 minutes. No tools are needed for assembly. Put a blanket underneath to protect your floor.

The rollator can be used by different people. Before use, check that the height is set correctly.

Important: Please check the rollator for transportation damage. Do not use the rollator if there is obvious damage. Contact us regarding repairs or return.

If you have any questions regarding the assembly, the rollator or if you are unsure about anything, please contact our customer service at the e-mail address: support@helavo-rollators.com

Scope of delivery

The Helavo model series H1030 is delivered with a frame and the following parts:

- Removable cup holder
- Basket
- Tray with anti-slip mat
- Wing screws
- Rear axles with rear wheels
- Front wheels
- Handle with parking brake
- Velcro strap
- User instructions
- Product insert

Description of the Rollator

H1030-Series rollators

Helavo model series H1030 consists of the following rollators:

- H1030 Indoor Rollator, black
- H1031 Indoor Rollator, white

Symbols

Manufacturer

Date of manufacture

Catalog No.

Batch code

Medical Device


Unique device identifier

Instruction for use

CE-Marking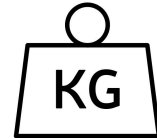

Maximum load capacity

Safety

Warning: The rollator is designed for a maximum load of 240 lbs. The maximum load of the basket of 11 lbs should not be exceeded.

Observe the following safety instructions

1. Ensure that the parking brake ("G") is active when the rollator is parked.
2. Check the handle height and make sure the height adjustments ("I") are properly locked before use.
3. Make sure the rollator is fully unfolded and locked into position with a "click" sound.
4. Keep your hands away from the folding mechanism to avoid entrapment.

What not to do

1. Do not sit on the rollator or use it like a wheelchair.
2. Do not hang objects on the brake handle ("H") or the handle ("A").
3. Do not stand on the rollator or use it as a stepladder.

If you have any questions regarding the assembly, the rollator or if you are unsure about anything, please contact our customer service at the e-mail address: support@helavo-rollators.com

Assembly instructions

1. Remove the rollator from the packaging and remove the protective foils. Spread out the parts for assembly as shown.

2. Remove the wing screws on the frame ("K") marked "RB" on the rollator.

3. Insert the rear wheel axle with rear wheel ("L") with the marking "RB" into the frame at the corresponding marking "RB". Both markings point to the outer side of the rollator.

4. Fasten the rear wheel axle with rear wheel ("L") with the previously loosened wing screw ("K").

Assembly instructions


5. Repeat steps 2. - 4. for the second rear wheel axle with rear wheel ("L") with the "LB" mark on the other side of the frame.

6. Remove the wing screw ("E") from the front wheel ("F") marked "RF".

7. Insert the front wheel ("F") into the frame at the corresponding mark "RF".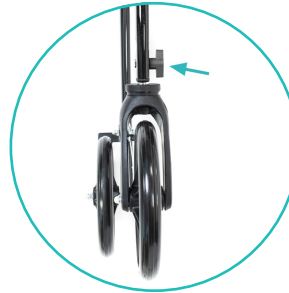

8. Fasten the front wheel ("F") with the previously loosened wing screw ("E"). The wing screw points inwards. When turning, the screw head tightens in the frame.

Assembly instructions

9. Repeat steps 6. - 8. for the second front wheel ("F") with the "LF" mark on the other side of the frame.

10. Put the rollator in an upright position to unfold it. To do this, pull the "Pull" button ("J").

Important: In order for the rollator to be completely unfolded, it must lock into position with a "click" sound.

11. Open the gray height adjustment ("I") on both sides of the frame.

12. Attach the handles and close the height adjustment ("I") so that both sides are at the same height.

Assembly instructions

- 13.** Our recommendation: Fix the brake cables on both sides of the frame using the velcro tape as shown.

- 14.** Place the tray with anti-slip mat ("B") on the frame.

Adjusting the height

Open the gray height adjustment ("I") and adjust the handle into an appropriate height. Repeat for the other side. Make sure the height adjustment on both sides are securely locked and closed. Make sure that the handle is set to the same height on both sides.

Handle height: 35.4 - 40.2 inch

Your ideal handle height

Adjust the rollator so that your arms are slightly bending when holding onto the handle ("A"). If you find that you need to bend over or keep your arms bent too much, adjust the height of the handles.

Removable cup holder

1. Attach the cup holder ("C") to the basket in the provided position.

2. To do this, slide the cup holder ("C") onto the bracket from above.

Brakes

1. Pull the brake handle ("H") upwards to brake the rollator.

2. Push the brake handle ("H") upwards and fix it with the parking brake ("G") to lock the rear wheels. In this position, the parking brake of the rollator is active. To release the parking brake, push the brake handle up slightly and pull the parking brake forward.

Adjusting the brakes

To adjust the rear brake ("M"), adjust the adjustment screw and locking nut. These are located on the rear wheels ("L") of the rollator. Turn the adjustment screw counterclockwise to increase the tension. Then secure the adjustment by turning the locking nut clockwise. Hold the adjustment screw in place while doing so.

! Important

If the brake adjustment has been made so that the rear brake ("M") is set too loose or too tight, this will affect the function of the brake. If the brake is adjusted too tight, it may drag on the wheel. If the brake is set too loose, it could affect the operation of the brake.

Folding and storing the rollator

1. Remove the tray ("B"). Then activate the parking brake ("G") (See also chapter "Brakes" point 2.).

2. Grasp the rollator by the handle ("A") and tilt it in your direction so that it stands on the rear wheels ("L").

3. Pull the "Pull" button ("J") so that the rollator folds up. If the rollator does not fold automatically, tilt it more.

4. To unfold the rollator again, lift it slightly by the front edge of the basket ("D"). **Important:** For the rollator to be completely unfolded, it must lock into place with a "click" sound.

Maintenance

1. Check the rollator periodically for loose wing screws ("E" / "K") and defective height adjustments ("I").
2. Periodically check for proper operation of rear wheel brakes ("M").
3. Replace damaged or worn parts.

Cleaning

The rollator can be cleaned with a damp cloth / sponge and warm soapy water. If necessary, you can use an antibacterial surface cleaner. Dry the areas thoroughly afterwards. When cleaning, make sure not to leave any residue on the handle so that the surface does not become slippery. Do not use scouring pads or scrapers to avoid damaging the surface coating.

Use and disposal

It is important that the rollator is inspected and serviced regularly to ensure that it is in good condition and safe to use.

Be aware of the following points:

1. Do not use the rollator if any parts are defective or missing. Especially screws or wheels. ("E"/"F"/"K"/"L").
2. Do not use the rollator if the wheels ("F"/"L") are defective.
3. Do not use the rollator if the height adjustment ("I") of the handles does not engage properly or is defective.
4. Do not use the rollator if the "Pull"-Button ("J") of does not work properly or is defective.
5. Do not use the rollator if the brake ("H"/"M") does not work properly.
6. Do not use the rollator if it does not engage properly when folded up.

Disposal

The disposal of a rollator should be done carefully to ensure that no pollution occurs and that all materials are disposed of properly.

Check the local regulations in your city or town to find out how to properly dispose of the rollator.

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