Rollator Series H1070

applies to H1070 - H1079

User instructions
Bedienungsanleitung
Mode d'emploi
Instrucciones de uso
Istruzioni per l'uso
Bruksanvisningar
Gebruiksaanwijzing
Instrukcja obsługi

 helavo

Languages

4		USER INSTRUCTIONS
25		BEDIENUNGSANLEITUNG
46		MODE D'EMPLOI
67		INSTRUCCIONES DE USO
88		ISTRUZIONI PER L'USO
109		BRUKSANVISNINGAR
112		GEBRUIKSAANWIJZING
115		INSTRUKCJA OBSŁUGI

Rollator Series H1070

English User instructions

Intended use

This rollator is intended to be used by persons with reduced mobility as a walking aid, as a wheelchair and for transporting smaller objects. This rollator model is designed for indoor and outdoor use. It is intended for use on solid ground by persons with a maximum weight of 300 lbs. Do not use the rollator in combination with other mobility aids.

The user should have the necessary physical, visual and cognitive abilities to operate the rollator safely.

Before using your rollator

Read the user instructions carefully **before** assembling and using the rollator and keep it in a safe place. Consult your medical professional to determine if the rollator is suitable for your needs.

You can assemble this rollator on your own in a few steps. Assembly takes about 10 minutes. No tools are needed for assembly. Put a blanket underneath to protect your floor.

The rollator can be used by different people. Before use, check that the height is set correctly.

Important: Please check the rollator for transportation damage. Do not use the rollator if there is obvious damage. Contact us regarding repairs or return.

If you have any questions regarding the assembly, the rollator or if you are unsure about anything, please contact our customer service at the e-mail address: support@helavo-rollators.com

Scope of delivery

The Helavo Series H1070 comes with a frame and the following parts:

- Removable cup holder
- Removable cane holder
- Bag
- Wing screws
- Front wheels
- Rear axle with rear wheels
- Legrests
- Velcro straps
- User instructions
- Product insert
- Step aid

Description of the rollator

H1070-Series rollators

Helavo model series H1070 consists of the following rollators:

- H1070 Hybrid-Rollator, black

Symbols

Manufacturer

Date of manufacture

Catalog No.

Batch code

Medical Device


Unique device identifier

Instruction for use

CE-Marking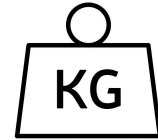

Maximum load capacity

Safety

Warning: The rollator can provide assistance to a person with a maximum weight of 300 lbs. The maximum load of the basket of 11 lbs should not exceeded.

Follow the safety instructions


1. Make sure the parking brake ("L") is active when the rollator is parked.
2. Check the handle height and make sure the height adjustment ("M") is locked before use.
3. Check the height of the Legrests and make sure the spring buttons ("G") and the screws are properly engaged and tightened before use.
4. Make sure the rollator is completely unfolded and locked into position with a "click" sound.
5. Keep your hands away from the folding mechanism to avoid entrapment.

What not to do


1. Do not sit on the rollator if the parking brake ("L") is not active or if it is at an angle.
2. Do not drop onto the seat ("D").
3. Do not hang any objects on the brake handles ("L") or hand grips ("K").
4. Do not stand on the rollator or use it as a stepladder.

If you have any questions regarding the assembly, the rollator or if you are unsure about anything, please contact our customer service at the e-mail address: support@helavo-rollators.com


Assembly instructions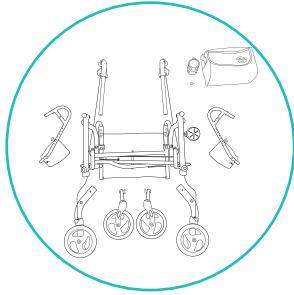


1. Remove the rollator from the packaging and remove the protective foils. Spread out the parts for assembly as shown. The seat ("D") of the rollator lies flat.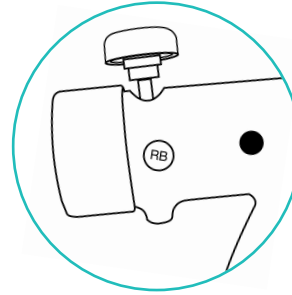

2. Remove the two wing screws ("N") on the frame of the rollator at the "RB" mark.

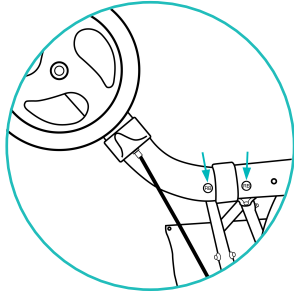
3. Now insert the rear wheel axle with rear wheel ("O") with the marking "RB" into the frame at the corresponding marking "RB". Squeeze the push buttons on both sides of the rear wheel axle.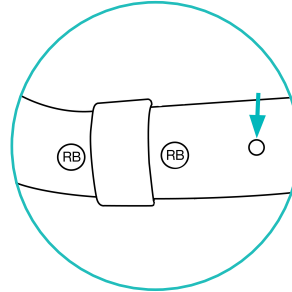


4. **Important:** Make sure that the rear wheel axle with rear wheel ("O") engages with a "click" sound.


Assembly instructions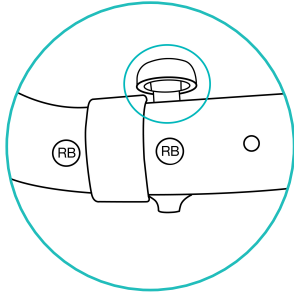


5. Fasten the rear wheel axle with rear wheel ("O") with the wing screws ("N") loosened before.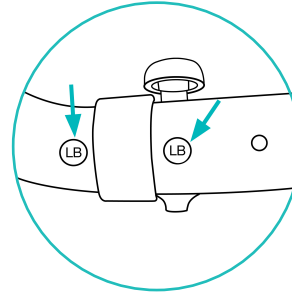

6. Now repeat steps 2. - 5. for the second rear wheel axle with rear wheel ("O") with mark "LB" on the other side of the frame.

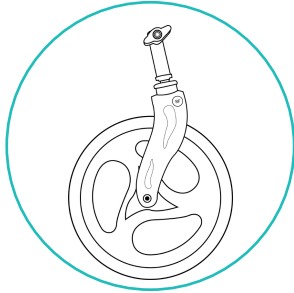
7. Remove the wing screw ("S") on the front wheel ("T") with the marking "RF".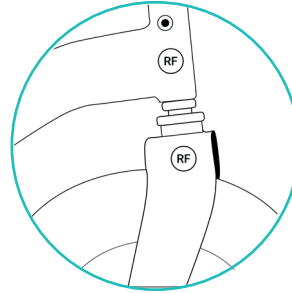


8. Insert the front wheel ("T") into the frame at the corresponding "RF" mark.


Assembly instructions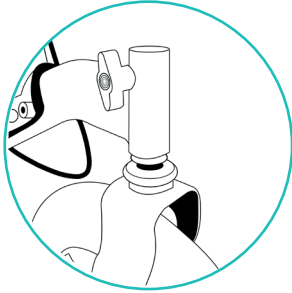


9. Fasten the front wheel ("T") with the previously loosened wing screw ("S"). The gray wing screw pointing outwards. When turning, the screw head will tighten in the frame.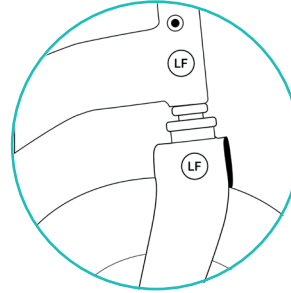

10. Repeat steps 7. - 9. for the second front wheel ("T") with the mark "LF" on the other side of the frame.

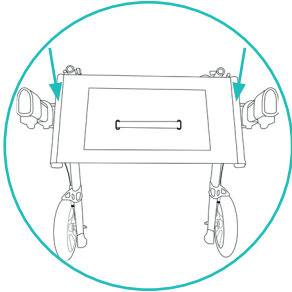
11. Put the rollator in an upright position and pull it apart by both handles ("K"). Push it down on both sides of the seat ("D") as shown. **Important:** For the rollator to be fully unfolded, it must lock into place with a "click" sound.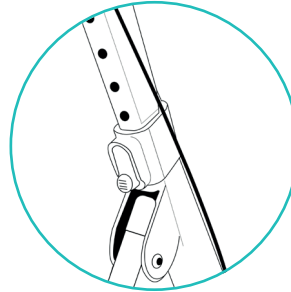


12. Press and hold the gray height adjustment button ("M") and insert the two handles ("K") into the frame one after the other. Adjust the handles so that both sides are the same height.


Assembly instructions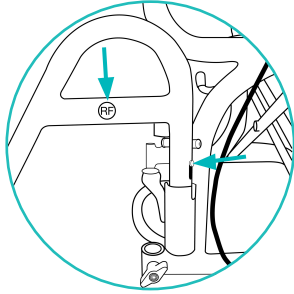


- 13.** Insert the legrest ("F") with the "RF" marking into the click mechanism ("J"). To do this, press in the golden pushbutton. (See also chapter: Adjusting and removing the legrests).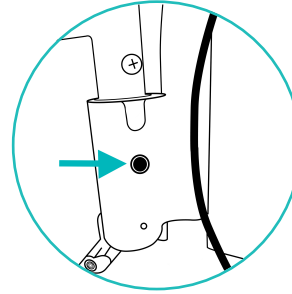

- 14.** Make sure that the legrest ("F") is engaged with a "click" sound.

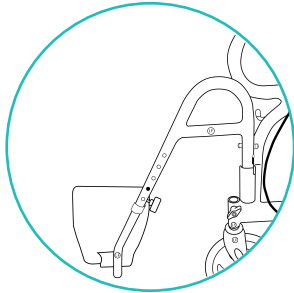
- 15.** Repeat steps 13. - 14. for the second legrest ("F") with the "LF" mark on the other side of the frame.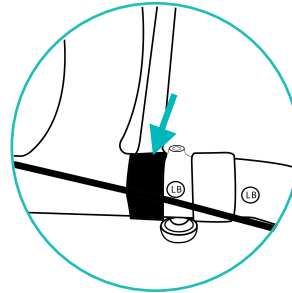

- 16.** Our recommendation: Fix the brake cables on both sides of the frame using the velcro tape as shown.

Assembly instructions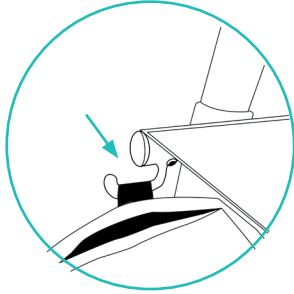

17. Attach the bag ("E"). Hook the tabs of the bag into the hooks provided.


Adjusting the height

Press the gray button on the height adjustment ("M") on the frame of the rollator and adjust the handle to the appropriate height. Repeat for the other side. Make sure the gray buttons on both handles are locked. Make sure that both handles are set to the same height.


Handle height: 32.7 - 37.8 inch

Your ideal handle height


Adjust the handles ("K") of the rollator so that they are at the height of your wrist when your arms are hanging. When you place your hand on the handle, your elbow should be slightly bent. If you find that you need to bend over or keep your arms bent at a right angle, adjust the height of the handles.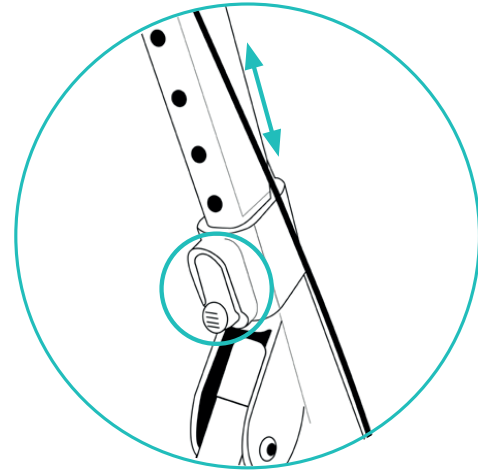

Brakes

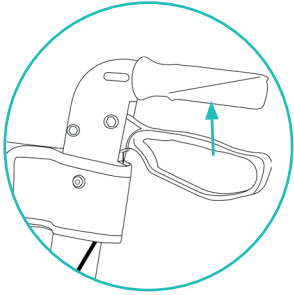
1. Pull the brake handles ("L") up to brake the rollator.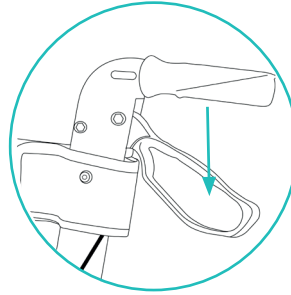

2. Push the brake handles ("L") down to lock the rear wheels. In this position, the parking brake of the rollator is active.
To release the parking brake again, pull the brake handles ("L") upward.

Adjusting the brakes


To adjust the rear brake ("P"), adjust the adjustment screw and locking nut. These are located on the rear wheels ("O") of the rollator. Turn the adjustment screw counterclockwise to increase the tension. Then secure the adjustment by turning the locking nut clockwise. Hold the adjustment screw in place while doing so.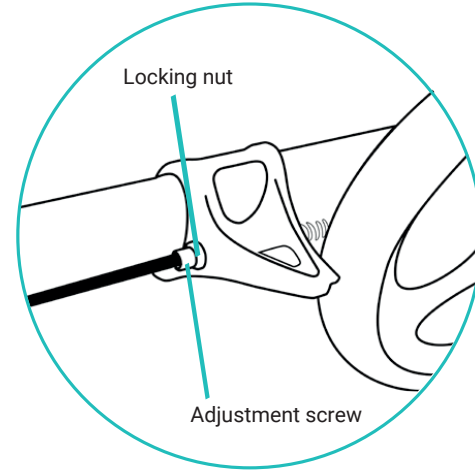


! Important

If the brake adjustment has been made so that the rear brake ("P") is set too loose or too tight, this will affect the function of the brake. If the brake is set too tight, it could drag on the wheel. If the brake is set too loose, it could affect the operation of the brake.

Removable cane holder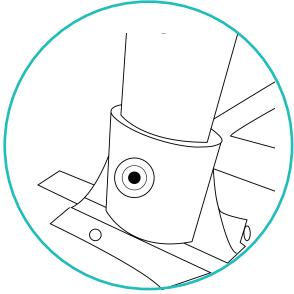


1. Attach the cane holder ("R") to the side of the frame in the provided position.

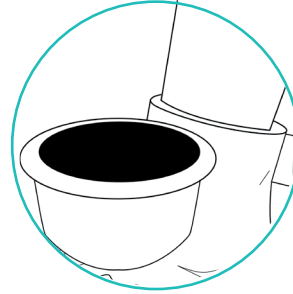
2. To do this, slide the cane holder ("R") onto the holder from above.

Removable cup holder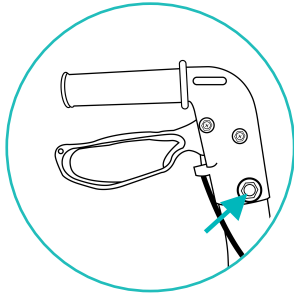


1. Attach the cup holder ("A") to the side of the handle in the provided position.

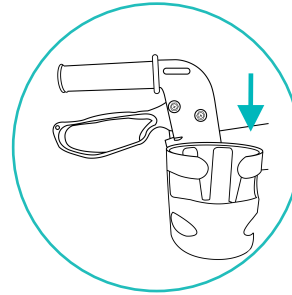
2. To do this, slide the cup holder ("A") onto the holder from above.

Folding and storing the rollator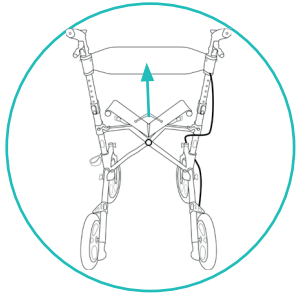

1. Pull the handle in the middle of the seat ("D") firmly upwards to fold the rollator.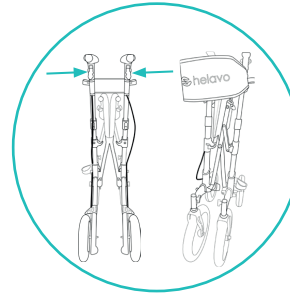


2. Continue to push the rollator on both sides of the frame.

Conversion of the rollator

When using the rollator in its conventional function as a rollator, make sure that:

1. The backrest ("B") faces the front wheels.
2. The legrests ("F") are folded in and the footrests ("I") are folded up.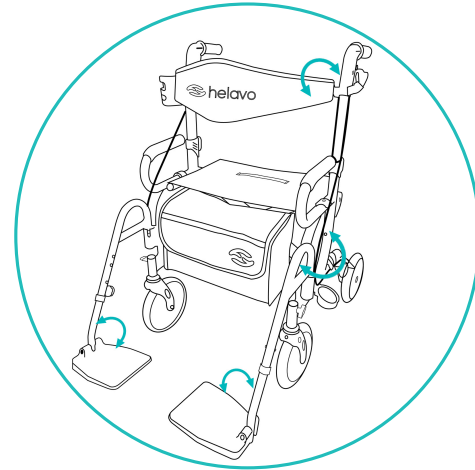

Converting the rollator for use as a wheelchair

1. Fold the backrest ("B") to the other side.
2. Lift the legrests ("F") slightly and fold them forward one by one. Make sure that the legrests are locked in place.
3. Fold down the footrests ("I") one after the other after you have sat down on the seat of the wheelchair ("D").


Important

Make sure you sit down safely on the seat ("D") and also stand up safely again. First sit down and then fold the footrests ("I") down. Fold the footrests up first and then stand up.

Adjusting and removing the legrests

Adjust the legrests ("F") to your suitable height. Make sure that both legrests are set to the same height

1. Loosen the locking screw ("H").
2. Press the pushbuttons ("G") on both sides and adjust the legrest to the desired length. Make sure that the legrest locks into place with a "click" sound.
3. Fix the legrest ("F") with the previously loosened locking screw ("H").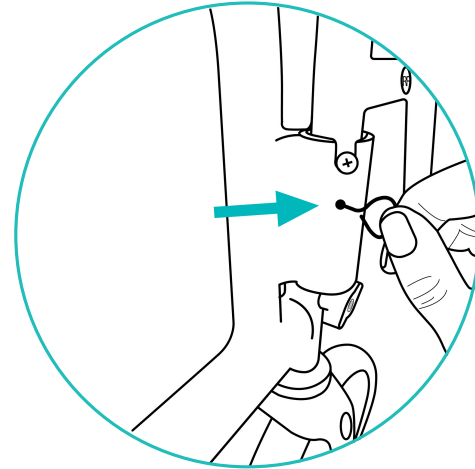


Removing the legrests

1. Fold the legrest ("F") forwards.
2. With the help of the silver clip, press the click mechanism of the legrest ("J") through the hole in the frame at the corresponding position.
3. Lift the legrest at the same time to lift it out of the frame.

Maintenance

1. Check the rollator at regular intervals for loose wing and locking screws ("H"/"N"/"S") and defective height adjustments ("M").
2. Periodically check for proper operation of the rear wheel brakes ("P").
3. Replace damaged or worn parts.

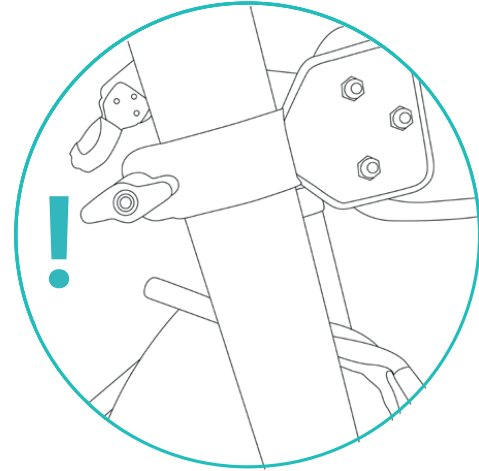Cleaning

The rollator can be cleaned with a damp cloth / sponge and warm soapy water. If necessary, you can use an antibacterial surface cleaner. Dry the areas thoroughly afterwards. When cleaning, take care not to leave any residue on the seat and handles so that the surface does not become slippery in these areas. Do not use scouring pads or scrapers to avoid damaging the surface coating.

Use and disposal

It is important that the rollator is inspected and serviced regularly to ensure that it is in good condition and safe to use.

Be aware of the following points:

1. Do not use the rollator if any parts are defective or missing. Especially screws or wheels ("H"/"N"/"O"/"S"/"T").
2. Do not use the rollator if the wheels ("O"/"T") are defective.
3. Do not use the rollator if the height adjustment ("M") of the handles does not engage properly or is defective.
4. Do not use the rollator if the brake ("L"/"P") does not work properly.
5. Do not use the rollator if it no longer engages properly when unfolded.

Disposal

The disposal of a rollator should be done carefully to ensure that no pollution occurs and that all materials are disposed of properly.

Check the local regulations in your city or town to find out how to properly dispose of the rollator.

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