
# Rollator Walker H1080

applies to H1080-H1089

---

User instructions  
Bedienungsanleitung  
Mode d'emploi  
Instrucciones de uso  
Istruzioni per l'uso  
Bruksanvisningar  
Gebruiksaanwijzing  
Instrukcja obsługi

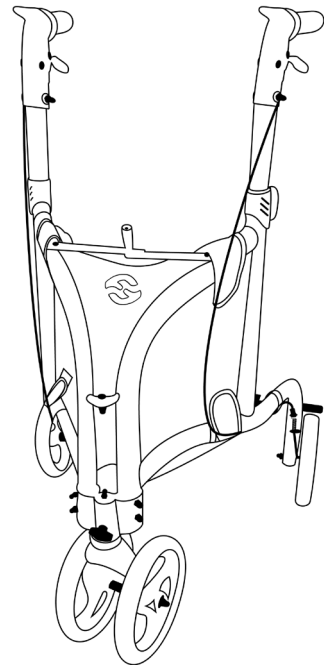 helavo

## User Instructions

4		USER INSTRUCTIONS
19		BEDIENUNGSANLEITUNG
34		MODE D'EMPLOI
49		INSTRUCCIONES DE USO
64		ISTRUZIONI PER L'USO
79		BRUKSANVISNINGAR
82		GEBRUIKSAANWIJZING
85		INSTRUKCJA OBSŁUGI

# Rollator Series H1080

## English user instructions

### INTENDED USE

This rollator is intended to be used by persons with reduced mobility as a walking aid and for transporting smaller objects. This rollator model is designed for indoor and outdoor use. It is intended for use on solid ground by persons with a maximum weight off 300 lbs. Do not use the rollator in combination with other mobility aids.

The user should have the necessary physical, visual and cognitive abilities to operate the rollator safely.

### Before using your rollator

Read the user instructions carefully before assembling and using the rollator and keep it in a safe place. Consult your medical professional to determine if the rollator is suitable for your needs.

You can assemble this rollator on your own in a few steps. Assembly takes about 10 minutes. No tools are needed for assembly. Put a blanket underneath to protect your floor.

The rollator can be used by different people. Before use, check that the height is set correctly.

**Important:** Please check the rollator for transportation damage. Do not use the rollator if there is obvious damage. Contact us regarding repairs or return.


If you have any questions regarding the assembly, the rollator or if you are unsure about anything, please contact our customer service at the e-mail address: [support@helavo-rollators.com](mailto:support@helavo-rollators.com)

## Scope of delivery

The Helavo Series H1080 comes with an aluminium frame and the following parts:

- Removable cup holder
- Bag
- Wing screws
- Front wheel
- User instructions
- Product insert

## Description of the rollator

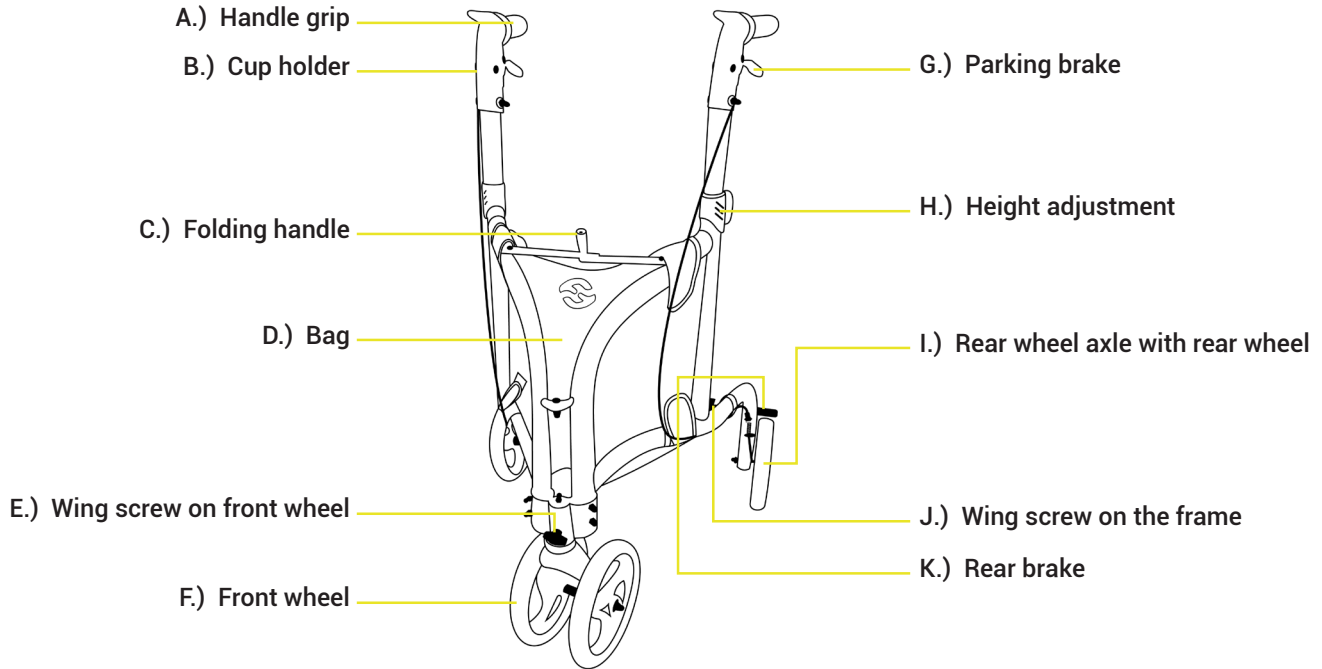## H1080-Series rollators

Helavo model series H1080 consists of the following rollators:

- H1080: Three Wheel Walker, black

## Symbols

Manufacturer

Date of manufacture

Catalog No.

Batch code

Medical Device


Unique device identifier

Instruction for use

CE-Marking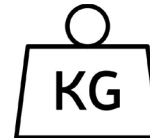

Maximum load capacity

## Safety

**Warning:** The rollator can provide assistance to a person with a maximum weight of 300 lbs. The maximum load of the bag of 11 lbs should not exceeded.

### Follow the safety instructions

1. Make sure the parking brake ("G") is active when the rollator is parked.
2. Check the handle height and make sure the height adjustment ("H") is locked before use.
3. Make sure the rollator is completely unfolded.
4. Keep your hands away from the folding mechanism to avoid entrapment.

### What not to do

1. Do not sit on the rollator.
2. Do not hang any objects on the brake handles ("G") or handle grips ("A").
3. Do not stand on the rollator or use it as a stepladder.

If you have any questions regarding the assembly, the rollator or if you are unsure about anything, please contact our customer service at the e-mail address: [support@helavo-rollators.com](mailto:support@helavo-rollators.com)

## Assembly instructions

1. Remove the rollator from the packaging and remove the protective foils. Spread out the parts for assembly as shown

2. Remove the wing screw on the rear axle with rear wheel („I“) with the marking „LB“. Now insert the rear wheel axle with rear wheel („I“) with the marking „LB“ into the frame at the corresponding marking „LB“. Both markings point to the outer side of the rollator.

3. Fasten the rear wheel axle with rear wheel („I“) with the wing screw („J“) loosened before. Attention: The wing screw must point to the inside of the rollator.

4. Now repeat steps 2. And 3. For the second rear wheel axle with rear wheel („I“) with mark „RB“ on the other side of the frame.

## Assembly instructions

5. Remove the wing screw („E“) on the front wheel („F“). Insert the front wheel („F“) and fasten it with the previous loosened wing screw („E“). Important: Make sure that the hole in the front wheel and the frame are flush, only then will the wing screw („E“) be able to be fitted correctly.

7. Press and hold the grey height adjustment button („H“) and insert the two handles („A“) into the frame one after the other. Adjust the handle so that both sides are the same height.

6. Put the rollator in an upright position and apart it by the folding handle („C“). Important: Make sure that the rollator is completely unfolded

## Adjusting the height

Press the gray button on the height adjustment ("H") on the frame of the rollator and adjust the handle to the appropriate height. Repeat for the other side. Make sure the gray buttons on both handles are locked. Make sure that both handles are set to the same height.

**Handle height:** 23.8 - 36.8 inch

### Your ideal handle height

Adjust the handles ("A") of the rollator so that they are at the height of your wrist when your arms are hanging. When you place your hand on the handle, your elbow should be slightly bent. If you find that you need to bend over or keep your arms bent at a right angle, adjust the height of the handles.

## Brakes

1. Pull the brake handles ("G") up to brake the rollator.

2. Push the brake handles ("G") down to lock the rear wheels. In this position, the parking brake of the rollator is active.  
To release the parking brake again, pull the brake handles ("G") upward.

## Adjusting the brakes

To adjust the rear brake ("K"), adjust the adjustment screw and locking nut. Turn the adjustment screw counterclockwise to increase the tension. Then secure the adjustment by turning the locking nut clockwise. Hold the adjustment screw in place while doing so.

### **!** Important

If the brake adjustment has been made so that the rear brake ("K") is set too loose or too tight, this will affect the function of the brake. If the brake is set too tight, it could drag on the wheel. If the brake is set too loose, it could affect the operation of the brake.

## Removable cup holder

1. Attach the cup holder ("B") to the side of the handle in the provided position.


2. To do this, slide the cup holder ("B") onto the holder from above.

## Folding and storing the rollator

1. Pull the folding handle („C“) towards you to fold up the rollator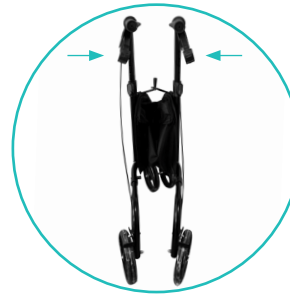

2. Continue to push the rollator on both sides of the frame.

## Maintenance

1. Check the rollator at regular intervals for loose wing screws ("E" / "J" ) and defective height adjustments ("H").
2. Periodically check for proper operation of the rear wheel brakes ("K").
3. Replace damaged or worn parts.

## Cleaning

The rollator can be cleaned with a damp cloth / sponge and warm soapy water. If necessary, you can use an antibacterial surface cleaner. Dry the areas thoroughly afterwards. When cleaning, take care not to leave any residue on the seat and handles so that the surface does not become slippery in these areas. Do not use scouring pads or scrapers to avoid damaging the surface coating.

## Use and disposal

It is important that the rollator is inspected and serviced regularly to ensure that it is in good condition and safe to use.

### Be aware of the following points:

1. Do not use the rollator if any parts are defective or missing. Especially screws or wheels ("E"/"F"/"J").
2. Do not use the rollator if the wheels ("F"/"I") are defective.
3. Do not use the rollator if the height adjustment ("H") of the handles does not engage properly or is defective.
4. Do not use the rollator if the brake ("E"/"J") does not work properly.
5. Do not use the rollator if it does not engage properly when folded up.

### Disposal

The disposal of a rollator should be done carefully to ensure that no pollution occurs and that all materials are disposed of properly.

Check the local regulations in your city or town to find out how to properly dispose of the rollator.

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