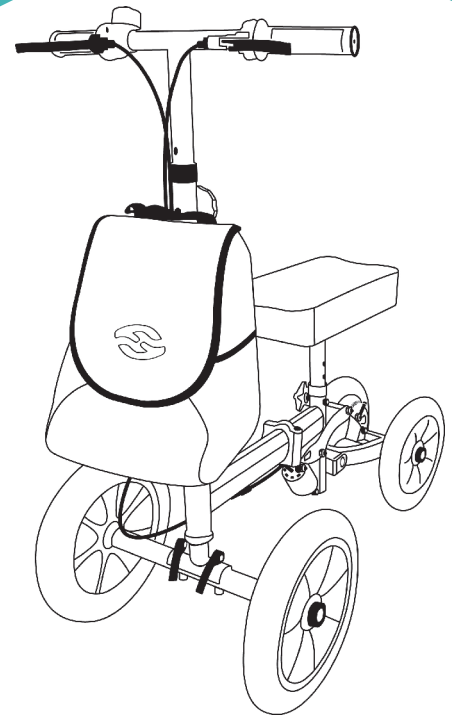
Knee Walker H1090

applies to H1090-H1099

User instructions
Bedienungsanleitung
Mode d'emploi
Instrucciones de uso
Istruzioni per l'uso
Bruksanvisningar
Gebruiksaanwijzing
Instrukcja obsługi

 helavo

User Instructions

4		USER INSTRUCTIONS
20		BEDIENUNGSANLEITUNG
36		MODE D'EMPLOI
52		INSTRUCCIONES DE USO
68		ISTRUZIONI PER L'USO
84		BRUKSANVISNINGAR
87		GEBRUIKSAANWIJZING
90		INSTRUKCJA OBSŁUGI

Knee Walker Series H1090

English user instructions

INTENDED USE

This knee walker is intended to be used by persons with reduced mobility as a walking aid and for transporting smaller objects. This Knee walker model is designed for indoor and outdoor use. It is intended for use on solid ground by persons with a maximum weight off 300 lbs. Do not use the knee walker in combination with other mobility aids.

The user should have the necessary physical, visual and cognitive abilities to operate the knee walker safely.

Before using your Knee walker

Read the user instructions carefully before assembling and using the knee walker and keep it in a safe place. Consult your medical professional to determine if the knee walker is suitable for your needs.

You can assemble this knee walker on your own in a few steps. Assembly takes about 10 minutes. No tools are needed for assembly. Put a blanket underneath to protect your floor.

The knee walker can be used by different people. Before use, check that the height is set correctly.

Important: Please check the rollator for transportation damage. Do not use the knee walker if there is obvious damage. Contact us regarding repairs or return.

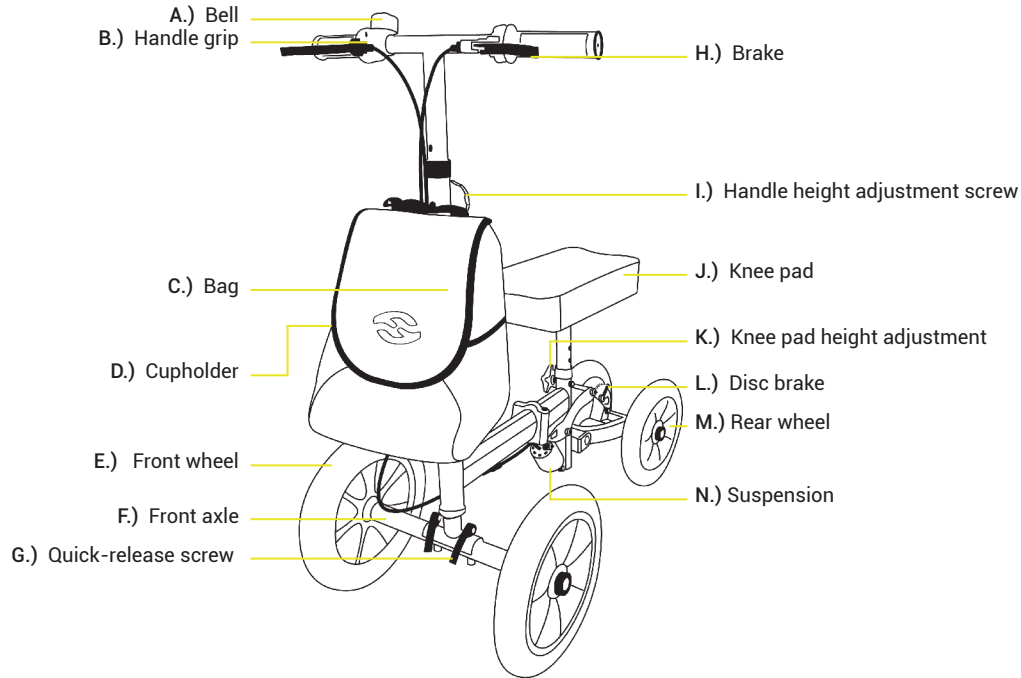
If you have any questions regarding the assembly, the knee walker or if you are unsure about anything, please contact our customer service at the e-mail address: support@helavo-rollators.com

Scope of delivery

The Helavo Series H1090 comes with an aluminium frame and the following parts:

- Removable cup holder
- Knee pad
- Bag
- Screws
- Front Wheels with axle
- User instructions
- Product insert
- Bell

Description of the knee walker

H1090-Series knee walkers

Helavo model series H1090 consists of the following knee walkers:

- H1090: Knee Walker, green

Symbols

Manufacturer

Date of manufacture

Catalog No.

Batch code

Medical Device

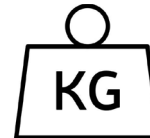
Unique device identifier

Instruction for use

CE-Marking

Maximum load capacity

Safety

Warning: The knee walker can provide assistance to a person with a maximum weight of 300 lbs. The maximum load of the bag of 11 lbs should not be exceeded.

Follow the safety instructions

1. Check the handle height and make sure the height adjustment ("I") is locked before use.
2. Make sure the knee walker frame and steering rode are completely fixed with the quick release screws.
3. Keep your hands away from the disc brake.

What not to do

1. Do not „drop“ onto the knee pad („J“).
2. Do not hang any objects on the brake handles ("H") or handle grips ("B").
3. Do not stand on the knee walker or use it as a stepladder.
4. Do not operate the model with more than one person.
5. Do not drive too fast.

If you have any questions regarding the assembly, the knee walker or if you are unsure about anything, please contact our customer service at the e-mail address:
support@helavo-rollators.com

Assembly instructions

1. Remove the knee walker from the packaging and remove the protective foils. Spread out the parts for assembly as shown.

2. Fold the steering rod upwards into an upright position. Make sure that the steering rod locks into place and is secure. Press the quick-release screw upwards into the recessed device.

3. Close the quick-release screw and press it tight so that the steering rod is securely locked.

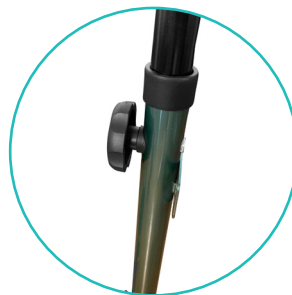
4. Unfold the frame of the scooter in the middle so that it locks into place when unfolded. Press the quick-release screw upwards into the recessed device.

Assembly instructions


5. Close the quick-release screw and press it tight so that the frame is securely locked.

6. Insert the handle bar and fasten it with the handle height adjustment screw („I“). Important: Make sure that the holes are flush, only then will the handle height adjustment screw („I“) be able to be fitted correctly.

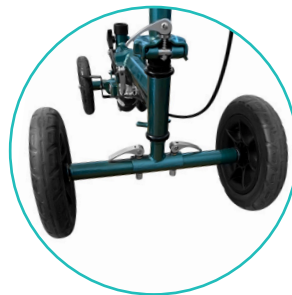7. Insert the front axle („F“) and fasten it with the quick-release screws („G“). Important: Make sure that the hole in the front wheel and the frame are flush, only then will the quick-release screws („G“) be able to be fitted correctly.


8. Fasten both quick-release screws („G“) and make sure that the front axle is well fixed to the frame.

Assembly instructions

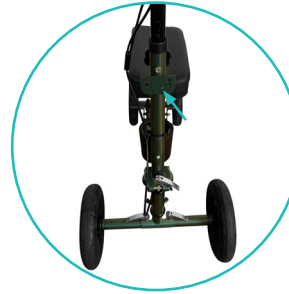9. Attach the knee pad („J“) and fix it with the knee pad height adjustment screw („K“) at the desired height. Make sure that the holes are flush, only then will the knee pad height adjustment screw („K“) be able to be fitted correctly.

10. Attach the bag („C“). Therefore slide the bag into the Mechanism on the handlebar.

Adjusting the handle height

Unscrew the handle height adjustment screw („I“). After loosened the screw, adjust the handlebar to the desired height and fix it with the previously loosened handle height adjustment screw. („I“). Make sure that the hole in the steering rod frame and the handle bare tube are flush, only then will the handle height adjustment screw („I“) be able to be fitted correctly.

Handle height: 32.2 - 40.5 inch

Adjusting the knee pad height

Unscrew the knee pad height adjustment screw („K“). After loosened the screw, adjust the knee pad to the desired height and fix it with the previously loosened knee pad height adjustment screw („K“). Make sure that the holes in the tubes are flush, only then will the knee pad height adjustment screw („K“) be able to be fitted correctly.

Knee pad height: 17.3 - 21.2 inch

Brakes

1. Pull the brake handles ("G") to brake the rollator.

Adjusting the brakes

To adjust the disc brake („L“), adjust the adjustment screw. Turn the adjustment screw counterclockwise to increase the tension.

! Important

If the brake adjustment has been made so that the disc brake („L“) is set too loose or too tight, this will affect the function of the brake. If the brake is set too tight, it could drag on the wheel. If the brake is set too loose, it could affect the operation of the brake.

Removable cup holder


1. Attach the cup holder („D“) to the steering rod frame in front of the knee pad.

2. To do this, slide the cup holder (“D“) onto the holder from above.

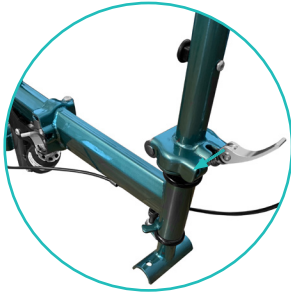Folding the handle bar and middle frame of the knee walker

1. Loosen the quick-release screw on the steering rod and bring it into the position shown in the picture. To fold the steering rod, pull the quick-release screw to the left so that the hooked bolt pops out of the mechanism and the handlebar can now be folded in.

2. Loosen the quick-release screw on the middle of the frame and bring it into the position shown in the picture. To fold the knee scooter in the middle of the frame, pull the quick-release screw upwards so that the hooked bolt pops out of the mechanism and the frame can now be folded.

Maintenance

1. Check the knee walker at regular intervals for loose screws ("G" / "I") and defective height adjustments ("I").
2. Periodically check for proper operation of the disc brake ("L").
3. Replace damaged or worn parts.

Cleaning

The knee walker can be cleaned with a damp cloth / sponge and warm soapy water. If necessary, you can use an antibacterial surface cleaner. Dry the areas thoroughly afterwards. When cleaning, take care not to leave any residue on the seat and handles so that the surface does not become slippery in these areas. Do not use scouring pads or scrapers to avoid damaging the surface coating.

Use and disposal

It is important that the knee walker is inspected and serviced regularly to ensure that it is in good condition and safe to use.

Be aware of the following points:

1. Do not use the knee walker if any parts are defective or missing. Especially screws or wheels ("E"/"G"/"I"/"K").
2. Do not use the knee walker if the wheels ("E"/"M") are defective.
3. Do not use the knee walker if the height adjustment ("I") of the handles does not engage properly or is defective.
4. Do not use the knee walker if the brake ("H"/"L") does not work properly.
5. Do not use the knee walker if it does not engage properly when folded up.

Disposal

The disposal of a knee walker should be done carefully to ensure that no pollution occurs and that all materials are disposed of properly.

Check the local regulations in your city or town to find out how to properly dispose of the knee walker.

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